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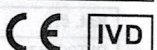
EN

## Intended use

Set for fast staining of blood smears.

## Principle

The set contains the solutions for fast staining of blood smears. Result coloration has panoptical character. Staining is performed by dipping smears into colouring solutions. Intensity and tinge of coloration can be changed by number of dips.

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## Reagents

- R1** Fixing solution  
Methanol 24.7 mol/l, Naphthalin green 1 mg/l
- R2** Staining solution 1  
Eosin Y 1.73 mmol/l, phosphate buffer, pH 6.8, 60 mmol/l
- R3** Staining solution 2  
Azur II 1.2 g/l, phosphate buffer, pH 6.8, 60 mmol/l
- R4** Tablets for rinsing solution  
phosphate buffer, pH 7.2, 2.5 mmol/tablet

## Storage

Store at (+2 to +25) °C

## Preparation of rinsing solution R4

In 50 ml of dist. water dissolve 1 tablet of Reagent 4.  
Stability: 2 months at (+2 to +25) °C in the dark.

## Procedure

- On fatless slides make a blood smears in a standard way. Let dry up free on the air.
- Pour the reagents into convenient staining cuvettes.
- Fix the smear by dipping 5-times on 1 sec. into Reagent 1. After every dip let the solution flow down and remove the excess by mopping of the drop against the wall of cuvette.
- Dip the fixed smear 3-times on 1 sec. into Reagent 2. After every dip let the solution flow down and remove the excess by mopping of the drop against the wall of cuvette.
- Dip the smear 6-times on 1 sec. into Reagent 3. After every dip let the solution flow down and remove the excess by mopping of drop against of wall of cuvette.
- Rinse the slide by rinsing solution and let dry up, on the air.

## Notes

- The intensity of coloration may be affected by number of dips into staining solutions. The minimal number of dips into each of them is 3.
- It is possible to increase the intensity of eosinophile coloration by increasing the number of dips into Reagent 2.
- It is possible to increase the intensity of blue coloration by increasing the number of dips into Reagent 3.
- All reagents are stable, when protected from direct sunshine. When coloration of standard is different from colouration of smears, change all solutions in staining cuvettes (cca 1-times of week).
- In case of pathological differential blood picture it is recommended to carry out pan-optical coloration according May-Grünwald and Giemsa-Romanowski. The coloration of the granules of basophiles with LEUCODIF is less remarkable. This staining is not usefull for thrombocytes.

## Health protection

For *in vitro* diagnostic use. To be handled by entitled and professionally educated person. Reagent 1 contains methanol (100%) which is toxic and flammable.

## Danger

### Hazard statement:

H225 Highly flammable liquid and vapour.  
H301+H311+H331 Toxic if swallowed, in contact with skin or if inhaled.  
H370 Causes damage to organs.

### Precautionary statement:

P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.  
P233 Keep container tightly closed.